INSTYLLA™

DELIVERY KIT

INSTRUCTIONS FOR USE

Instylla, Inc.
201 Burlington Road
Bedford, MA 01730
United States
Tel: +1 781-790-4860
www.instylla.com

80-3001-021 rev. B
Effective 2023-02-17

| INSTRUCTIONS FOR USE

This Page Left Intentionally Blank

Instylla Delivery Kit

Rx Only

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Device Description

The Instylla Delivery Kit is comprised of two delivery syringes, a syringe holder, and a plunger clip. Each syringe barrel contains a fixed male luer connector, which is compatible with female luer hubs.

Contents

<i>Item</i>	<i>Description</i>	<i>Quantity</i>
A	Syringe (Light Blue) - Labeled ①	1
B	Syringe (Yellow) - Labeled ②	1
C	Syringe Holder	1
D	Plunger Clip	1

Intended Use / Indications for Use

The Instylla Delivery Kit is intended to be used for the intra-arterial or intravenous administration of radiographic contrast media, saline and other aqueous solutions.

Contraindications

None known.


Precautions

- For single-use only, do not re-sterilize. Attempts to reuse or re-sterilize this device may compromise the structural integrity of the device and/or lead to device failure, which in turn may result in patient injury, illness, or death. Reusing, reprocessing, or re-sterilizing may also create a risk of contamination of the device and/or cause patient infection or cross infection including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death.
- Dispose of the used device safely, following your institution's local procedure for the disposal of medical waste.
- Do not use the syringes with syringe holder and plunger clip if any component appears cracked or broken.

MD	Medical device Producto sanitario Dispositif médical Dispositivo medico Medizinprodukt Medisch hulpmiddel	5.7.7.
UDI	Unique device identifier Identificador único del dispositivo Identifiant unique du dispositif Identificativo unico del dispositivo Einmalige Produktkennung Unieke hulpmiddelidentificatie	5.7.10.

Instructions for Use

1. Remove the syringes, syringe holder, and plunger clip from the packaging using sterile technique.
2. Fill the syringes with desired volumes of fluid using care that no air is introduced into the syringe.
3. Connect the yellow syringe to its desired lumen.
4. Connect the blue syringe to the second lumen.
5. Assemble the syringes to the syringe holder.
6. Attach the plunger clip to both plungers to couple the two together.
7. Delivery Kit is ready for use.

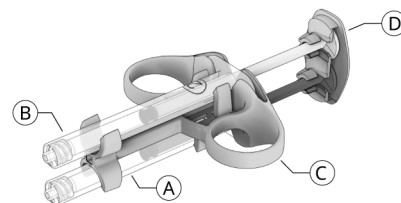8. Manually advance the plunger clip to deliver the fluids.
9. The Delivery Kit may be refilled for subsequent uses in the same procedure (single patient).
10. To refill Delivery Kit, follow the procedure below:
 - a. Disconnect the plunger clip.
 - b. Remove the syringes from holder.
 - c. Disconnect the syringes from their respective lumens.
 - d. Refill the syringes.
 - e. Reassemble the delivery kit as described above.

Nominal capacity accuracy per ISO 7886-1.

How Supplied

Supplied sterile.

Store in a cool, dry place.

Do not use if package is opened or damaged.

Do not use if labeling is incomplete or illegible.

Serious Incidents

If in relation to using this product, death or a serious deterioration in the state of health has occurred, please report the incident to the manufacturer and the competent authority of your country.

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